Quick Start Guide

Hear Audio

Insert Eko EARPIECE into the audio jack and twist to secure.

DUO can also be used with most 3.5mm headphones.




Turn DUO On/Off

Press and hold the center button for 1 second, then release.



Sounds + ECG


DUO is equipped with a digital stethoscope diaphragm and 2 ECG electrodes.

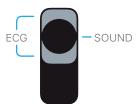

Placement

DUO streams a single lead ECG. Best placement varies. ECG must be placed upon the skin.

DUO can also be used to auscultate at all anatomical positions.

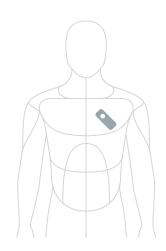

Link and Capture

Download the Eko app, available on the App Store® and follow the on-screen instructions to connect to DUO.







Volume Controls

Click the side buttons to adjust body sound amplification in the EARPIECE.



Change Audio Filter

Click the center button three times quickly.

The light ring will indicate the active audio filter.



(Pulmonary)

Click the center button twice to begin a recording.

Record Sounds and ECG

DUO must be paired to the Eko app to record.



Charging

Place the device on the wireless charging pad. Power the charging pad using the included micro USB cable and wall adapter. When the DUO is fully charged, all the lights will turn off on the charger and DUO.





DUO is on. Seeking device.



DUO is on. Paired with device.



DUO is on.

20% Battery.



DUO is on.

10% Battery.



DUO is

charging.



DUO is fully charged.

Learn More

DUO user manual available at ekohealth.com/getstarted or contact us at support@ekohealth.com

TM and © Eko Devices Inc. LBL 008 Rev G